

Product Data Sheet

NewPro Emplastrum

The re-usable, hemostatic wound dressing

You cut yourself shaving or doing housework? – No problem, just apply

NewPro Emplastrum onto the wound and the bleeding stops and a skin develops. The wound heals.

NewPro Emplastrum consists of high-quality polymer plastic. The material is tested for bio-compatibility and skin tolerance. **NewPro Emplastrum** is a medicinal product without chemical additives, which can be used for compressive wound treatment for small cuts and abrasions.

NewPro Emplastrum is meant to be used by one person only and can be re-used according to handling instructions and after a thorough cleaning and disinfection until its expiration date (for up to 5 years after production date).

NewPro Emplastrum
Closes small wounds in seconds
No burning pain, no wetting, no scab

Can be re-use multiple times

CE

REF FST-80/50

LOT 109893

07/2017

PZN - 9094793

For use by one person only

Decorated with the ihoch3-innovation-prize of the Federal Ministry of Education and Research

NewPro Emplastrum Application:

1. Do not wipe off blood.
2. Press foil onto wound so that the wound margin is covered.
3. Apply with pressure between 1 and 3 min. (depending on the size of the wound and blood clotting factor. Clean wound afterwards and clean and disinfect foil according to instructions.

